# Genius

GenOx Org

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#### GenOx Org Esponjoso / GenOx Org Cortical

##### DESCRIPTION

Exerto Ósseo Baumer - Orgânico (GenOx Org) is a resorbable implantable medical product composed of lyophilized demineralized bovine bone. Exerto Ósseo Baumer - Orgânico (GenOx Org) was designed to be used as an osteoconductive bone substitute for filling bone defects by dental and medical professionals. Exerto Ósseo Baumer - Orgânico (GenOx Org) is presented as opaque yellowish-white granules or blocks, the granules being presented in single glass bottles packed in blisters and the blocks packed in double blisters. Exerto Ósseo Baumer - Orgânico (GenOx Org) is a product of organic composition with a high degree of purity, free from contamination with heavy metals. It is an acellular, non-cytotoxic, non-immunogenic, non-pyrogenic and biocompatible product. Exerto Ósseo Baumer - Orgânico (GenOx Org) does not provide structural support, and there is recommended for filling in stable bone defects closed by walls, such as cavities and bone cysts, except for processes where its use is a diagnosis of uncontrolled cell growth. Electron microscopy studies show the presence of pores in the Exerto Ósseo Baumer - Orgânico (GenOx Org), which makes cells with osteogenic potential easier to deposit and adhere to the material, and consequently, a better condition for proliferation and necessary morphological maturation. For bone matrix production. Exerto Ósseo Baumer - Orgânico (GenOx Org) can be used alone or in combination with autogenous bone, or even associated with other bone substitute biomaterials of synthetic or natural origin (such as deproteinized bone or recombinant human bone morphogenetic protein or bovine). Exerto Ósseo Baumer - Orgânico (GenOx Org) is presented in two models: GenOx Org Cortical and GenOx Org Cancellous which are produced through the application of the same standard physico-chemical process for removing the mineral portion (decalcification) of young bovine bones. Exerto Ósseo Baumer - Orgânico (GenOx Org) is commercialized STERILE, being STERILIZED with Gamma RADIATION. Exerto Ósseo Baumer - Orgânico (GenOx Org) is a single use product, being PROHIBITED to REPROCESS. For greater patient comfort and safety, adhesive labels containing information for product traceability (such as batch number, expiration, etc.) are available inside the final packaging.

##### FUNDAMENTALS OF THE PRODUCT OPERATION AND ACTION

The production process of Exerto Ósseo Baumer - Orgânico (GenOx Org) preserves the original architecture and structural characteristics of the bone while maintaining the organic portion of the bone, allowing the final product to act as an osteoconductive element at the implant site. Exerto Ósseo Baumer - Orgânico (GenOx Org) is a biocompatible, acellular, non-cytotoxic, non-immunogenic, non-pyrogenic and high-purity bone substitute, as determined in vivo and in vitro studies. Due to its osteoconductive action and porosity, it acts as a support for deposition of new bone being reabsorbed and leaving new bone in place, allowing this product to be used in isolation or associated with other graft materials that have osteoinductive or osteogenic activity. Due to its chemical characteristics (organic nature), Exerto Ósseo Baumer - Orgânico (GenOx Org) is more quickly absorbed than totally deproteinized xenografts, being absorbed and integrated during the normal bone repair process by providing cells with osteogenic potential. Adhesion more favorable and more appropriate conditions to proliferate and finish its morphological maturation and finally produce bone matrix.

##### CONTENTS AND PRESENTATION

Exerto Ósseo Baumer - Orgânico (GenOx Org) is presented lyophilized and STERILE, the product being in the form of granules packaged in a glass bottle and presented in the form of blocks packaged in a double blister. The flask containing the product in granules or the blister containing the product in blocks is packed inside a blister (secondary packaging - plastic blister sealed in surgical grade paper) disposed in a cardboard box - final packaging. The box is labeled with the relevant data and information and laminated with heat shrinkable plastic. The package insert for the medical product (instructions for use) is available inside the final packaging - box.

Self-adhesive labels (five) for product traceability are available inside the final packaging. The following information and traceability items are found on the labels:  
Product model, technical name and trade name, Manufacturer's logo and address, Content description, Quantity and product size data, Product reference code, Production batch, Manufacturing date, Shelf life, Product registration number, Technical manager.

It is mandatory for the dentist or doctor to attach a label (label No. 1) to the patient's medical record or equivalent document, in order to comply with the current health legislation. In order to continue the traceability cycle, the manufacturer recommends that the health professional assign the other available labels (labels nº2 to nº5) as follows:  
Label Nº2: deliver to the patient  
Label Nº3: fix in the tax documentation that generates collection (AIH in the case of a SUS patient, or invoice in the case of a patient assisted by the complementary health system).  
Label Nº4: forward to supplier control  
Label Nº5: forward to the control of the responsible (main) surgeon.

It is the responsibility of the hospital institution to ensure the identification and traceability of the medical device at the time of use through the adhesive label provided.

Figura 1: General presentation of Exerto Ósseo Baumer - Orgânico (GenOx Org) in the form of granules in the foreground of the illustration, and detail of the product in blocks and the final packaging (box on the left).

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The product Exerto Ósseo Baumer - Orgânico model GenOx Org Cortical, which is composed of bovine bone of cortical origin demineralized by a standardized process of removing the mineral portion of the bones, is presented in the form of granules or blocks. The granules are made available to the market in granulometries from 0.25 mm to 9.0 mm in doses from 0.5cc (0.25g) to 400.0cc (200.0g), with each cubic centimeter (cc) equivalent to the weight approximately 0.5 gram. The blocks are presented in three dimensions in packages containing one to eight blocks each. Below are the codes for the product presentations. The product Baumer Bone Graft - Organic model GenOx Org Spongy, which is composed of bovine bone of spinal (spongy) origin by a standardized process of removing the mineral portion of the bones, is presented in the form of granules or blocks. The granules are made available to the market in granulometries from 0.25 mm to 9.0 mm in doses from 0.5cc (0.1g) to 400.0cc (80.0g), with each cubic centimeter (cc) equivalent to the weight approximately 0.2 gram. The blocks are presented in three dimensions in packages containing one to eight blocks each. Below are the codes for the product presentations. Below are tables of the product Exerto Ósseo Orgânico Baumer Model: GenOx Org Cortical presentation in granules containing the product codes for presentations by granule size, their doses in cc and weight in grams.

Exerto Ósseo Baumer - Orgânico			
GenOx Org Cortical grânulos de 0.50 to 1.0mm			
Model	Code	Dose (cc)	Weight (Aprox. g)
GenOx Org Cortical	931.50.0.5	0.5cc	0.3g
GenOx Org Cortical	931.50.1	1.0cc	0.5g

The information regarding the use, safety, handling, transport, storage and traceability of the product Exerto Ósseo Baumer Orgânico is presented as symbols as described in the chart below:

SYMBOL	MEANING
	single-use product
	Caution, consult accompanying documents
	Fragile, handle with care.
	Maximum temperature.
	Keep dry.
	Date of manufacture.
	Valid until.
	Batch Code.
	Número de catálogo.
	Sterilized by Gamma Ray.
	Do not use if package is damaged.
	CAUTION: FEDERAL LAW IN USA AND CANADA RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

**NOTE:** The graphic symbols related to the labeling of the medical product, referenced above, meet the requirements established in ISO 15223: 2000 Medical devices - Symbols to be used with medical device labels, labelling, and information to be supplied; ISO 15223: 2000 / Amd 1: 2002; and ISO 15223: 2000 / Amd 2: 2004.

##### SUPPORT MATERIAL

Support materials that come with the medical device are:  
- Product use instructions.  
- Identification label for protocol and patient

##### COMPOSITION

Baumer Bone Graft - Organic (GenOx Org) is a material of organic nature, product of the demineralization of cortical or medullary bovine bone by chemical and thermal processing until obtaining granules or blocks. Baumer Bone Graft - Organic (GenOx Org) is a product of an organic nature and in the monitoring of the physicochemical characteristics of Baumer Bone Graft - Organic (GenOx Org) the groups of components determined are presented in the following percentages: Fat = < 1%, Protein => 90%, Ash = < 5%, pH = 6-8. Heavy metals As = < 3ppm, Cu = < 3ppm, Cd = < 3ppm, Cr = < 3ppm, Hg = < 5ppm, Pb = < 30ppm.

##### INDICATIONS

Exerto Ósseo Baumer Orgânico (GenOx Org) is indicated for use as a bone substitute of osteoconductive action for filling bone defects by dental and medical professionals. Exerto Ósseo Baumer Orgânico (GenOx Org) was designed to be used in cavity filling procedures where there has been bone loss and which needs to be replaced by bone tissue. The product must be used in places that have three bone walls allowing physical support to the material and maintenance of the material in place. This product was not designed to be used in the areas of plastic and aesthetic surgery as a material for subcutaneous filling. This product can be used alone or associated with other bone substitute materials such as autogenous or autologous grafts. This product should not be used alone or associated with other bone substitute

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products in situations where there is uncontrolled cell growth (for example, cysts or tumors with suspected or diagnosed malignancy). Enxerto Ósseo Baumer Orgânico (GenOx Org) acts as an osteoconductive element functioning as biological support for cell development and growth during the bone regeneration process. The product of medullary origin, because it has a more porous and trabeculated architecture, has less resistance and time to resorption than the product of cortical origin.

Due to its organic nature, Enxerto Ósseo Baumer Orgânico (GenOx Org) is reabsorbed by the organism in less time than bone substitutes of inorganic nature in general (for example, deproteinized bone graft).

Enxerto Ósseo Baumer Orgânico (GenOx Org) must be hydrated before use, being mixed with the patient’s blood or sterile saline in order to facilitate handling the application of the product in the area to be filled.

#### IMPORTANT INFORMATION

Only qualified professionals with knowledge of the surgical techniques and procedures required for the proper choice, application and use of the product should use it.

The use of the product with surgical techniques and unsuitable conditions may harm the patient, leading to unsatisfactory results. The patient must undergo preoperative evaluation so that the best set of materials is determined and the most appropriate techniques are established for using the product.

Ideally, the surgeon should establish a post-operative control protocol including the measures taken in the situations found. It is also recommended that the surgeon keep a follow-up report of the evolution of the case together with the results

The surgeon must adopt classic prophylactic treatment described in the literature for immediate and late pre- and postoperative periods. The adhesive label accompanying the product must be attached to the patient’s protocol to be kept in the clinic or hospital.

#### TECHNICAL AND OPERATIONAL PREPARATION FOR USE OF THE PRODUCT.

The patient must undergo preoperative evaluation so that the best set of materials is determined and the most appropriate techniques are established for using the product.

The surgeon must adopt classic prophylactic treatment described in the literature for immediate and late pre- and postoperative periods. The surgeon should check the integrity of the packaging - Do not use if the packaging is violated.

The surgeon should check the product’s expiration date - Do not use if it is out of date

The surgical team should be responsible for preparing the patient and the area to perform a sterile procedure.

The surgical team should be responsible for preparing sterile instruments and support materials (including saline to hydrate the products).

The team should be responsible for observing the patient’s and team’s attire for sterile procedure.

#### INSTRUCTIONS FOR USE

Before opening the package, a container for hydrating Baumer Bone Graft - Organic (GenOx Org) must be prepared and combined with other graft materials. The container must be sterile as well as the fluid used for hydration. It is suggested to use sterile saline or blood of the patient removed in sterile conditions at the time of surgery.

- Observe preparation of the area and use of proper sterile dressing by the doctor, advisors and patient for sterile procedure.
- Check the integrity and validity of the product before opening
- Remove the vial from the final and secondary packaging.
- Using sterile gloves open the primary blister only in the area prepared for the surgical procedure.
- Remove the rubber seal and toggle.

**Attention:** Remove the seal carefully, the metal of the seal may pierce the gloves - Add the contents of the bottle to a sterile vat. In the case of use associated with other materials, add the product (s) in the sterile vat and mix with other synthetic or natural bone substitutes such as autogenous graft, autologous bone graft, lyophilized demineralized bone matrix (DFDB - human or bovine) etc.

- Mix sterile saline solution to the graft with a sterile spatula.

- The product will undergo an increase in volume due to hydration.

Attention: Before applying the product, prepare the area by debriding it with fibrous, necrotic or fibrous tissue and, in the case of direct contact with bone, perform decortication in order to provide direct contact of the graft with viable bone tissue.

- Irrigate the surgical site before placing the graft. For best results, do all site manipulation before graft placement.

- Use the product immediately after reconstitution with saline. Fill the bone gap with a sterile spatula.

Attention: ensure that there is adequate hemostasis to ensure that the material will be retained at the graft site.

- Proceed as usual to close or cover the implanted area.

Figura 3: Product detail Organic Graft Baumer Bone (GenOx Org) in the hydration phase of the materials to be grafted immediately before application to the bone defect.

#### INSTRUCTIONS AND PROCEDURES FOR DISPOSAL OF THE PRODUCT

The product Enxerto Ósseo Orgânico Baumer (GenOx Org) is for SINGLE USE, therefore what is not used (leftovers) must be properly mischaracterized and discarded by the hospital or dental user. The leftovers must be autoclaved before the final de-characterization by applying a disposal cycle or with equivalent lethality (eg 121 ° C for 30 minutes or 132-134 ° C 18-14 minutes).

The product whose packaging is damaged or tampered with (compromising the sterility of the product) or whose expiration date must NOT be used. The same procedures used to dispose of product leftovers must be followed.

BAUMER recommends that products in the situations described above be mechanically deformed, by rendering the remains unusable by destroying the glass bottle, cutting or tearing the primary, secondary and final packaging (box) and then they must be clearly and clearly identified. Visible, indicating its situation, that is, UNFIT FOR USE. The methods and procedures for disposing of the medical product used must ensure its complete lack of character, preventing any possibility of reuse. The mischaracterization of the medical product is the sole responsibility of the medical surgeon or dentist who is using the product, as well as the disposal methods and procedures used.

#### CARE AND PRECAUTIONS

For safety and effectiveness in the use of the Enxerto Ósseo Orgânico Baumer (GenOx Org) the following precaution and care measures must be observed:

- Every patient should be alerted to the risks associated with the functional use of the grafted area (chewing) until the complete bone neoformation is evidenced
- Only qualified professionals with knowledge of the surgical techniques and procedures necessary for proper choice and application should use the product
- The medical surgeon or dentist must be familiar with the procedures and recommendations for the implantation of (see INSTRUCTIONS FOR USE).
- Only specific sterile surgical instruments should be used.
- The surgeon must observe the conditions of preparation and asepsis of the area and instruments, and vestments of the team.
- As in all surgical procedures, care should be taken when treating patients medically compromised as patients on steroid therapy for a long time, or patients on therapy with anticoagulants.

Enxerto Ósseo Orgânico Baumer (GenOx Org) is a SINGLE USE PRODUCT, it is PROHIBITED to REPROCESS and should not be re-sterilized. Open products must be discarded as described above in this instruction for use.

#### WARNINGS

The surgeon must fully understand not only the medical and surgical aspects of the medical device.

The surgeon must take care to alert or test the patient for sensitivity to products based on bovine derivatives.

Smokers, drug users, alcoholics or malnourished patients should be alerted to the increased incidence of bone non-union.

**Note:** All bovine bone substitutes from the Genius line are manufactured with animal bones from the Brazilian herd. According to the geographical risk framework issued by the International Zoonansitary Code and by the European Community Scientific Steering Committee (SSC EC August 2005), Brazil is free from Bovine Spongiform Encephalopathy (BSE).

Although through decree 516/97, Brazil declares itself free from Bovine Spongiform Encephalopathy, although the processing to which the products are submitted is known to be effective in inactivating the causative agent of the EEBs, although the animals used for the production of the Genius line are registered in the Brazilian system of bovine and buffalo traceability - SIBOV, like all products of bovine origin, even if remote, there is a risk of transmission of BSE. In cases of symptoms of ncVD1 the health professional must notify the health authority (tecnovigilância. @ Anvisa.gov.br).

#### POSSIBLE ADVERSE EFFECTS

No adverse reactions have been reported for the product Organic Baumer Bone Graft (GenOx Org).

Possible complications such as swelling, infections, vascular disorders, scarring due to the access route, pain and complications associated with anesthesia may occur because they are common to implant surgery. As with any type of implant surgery, the patient may experience discomfort in the first postoperative days. Some pressure on the skin or gums may be observed / occur due to graft material with inadequate soft tissue coverage including the possibility of complete or partial extrusion.

In addition to those mentioned, normal reactions in periodontal surgeries can be found as: swelling of intraoral tissue, thermal sensitivity, gingival recession, some loss of height of crystal bone, etc. In clinical studies evaluating the product Enxerto

Ósseo Orgânico Baumer (GenOx Org), there were no immune reactions or hypersensitivity reactions. The surgeon must be fully informed about the medical and surgical aspects of the medical device. The surgeon must take care to alert or test the patient for sensitivity to products based on bovine derivatives. Smoking, drug users, alcoholics or malnourished and immunosuppressed patients should be warned about the increased incidence of bone non-union.

#### SPECIAL PRECAUTIONS AND CLARIFICATIONS ABOUT STORAGE AND TRANSPORTATION

The packaging must be intact upon receipt.

Opened packages must not be used (do not use the medical product if the packaging is broken);

The product must be transported and stored at room temperature. Avoid humidity and excessive heat;

The storage location of the medical device must be clean, dry and bright in order to maintain the ideal conditions for storage and transport, as well as the product’s physical and chemical integrity.

The carrier must be informed of the content and delivery time. The precautions to be taken during transport are demonstrated on the packaging, thus ensuring protection of the medical product from dispatch to delivery to the customer.

#### CONTRAINDICATIONS

The Baumer Organic Bone Graft product (GenOx Org) is not designed for any use other than that indicated.

The use of Enxerto Ósseo Orgânico Baumer (GenOx Org)in plastic and aesthetic surgery as material for subcutaneous filling is not recommended.

Do not use the product in the presence of any of the contraindications listed below or in the other known and / or reported contraindications in the general or specific literature.

The use of Enxerto Ósseo Orgânico Baumer (GenOx Org)is not recommended in patients with the following clinical conditions:

- Signs of local inflammation
- Uncontrolled metabolism disorders
- Mental disease
- Immunosuppressive pathologies
- Hypersensitivity or allergic reactions to collagen
- Hypersensitivity or allergic reactions to bovine derivatives
- Generalized renal dysfunction
- Severe liver disease
- Bone immaturity;
- Osteopenia or severe osteoporosis;
- Presence or recent history of systemic or localized infection (contaminated wounds in or near the implant)
- Alcoholism or drug abuse;
- Pregnancy and lactation (at the doctor’s discretion);
- Fewer or leukocytosis;

The use of the Enxerto Ósseo Orgânico Baumer product (GenOx Org) is not recommended in situations where periodontal surgery is not indicated.

The use of the Product Enxerto Ósseo Orgânico Baumer (GenOx Org) is not recommended in situations where there is no soft tissue for adequate covering of the graft used (from the grafted area).

The use of the Product Enxerto Ósseo Orgânico Baumer (GenOx Org) is not recommended for patients who are unwilling or unable to follow the post-operative care and instructions.

Enxerto Ósseo Orgânico Baumer (GenOx Org) should not be used in the presence of tumors or a history of malignancy in use associated with bone substitutes of osteoinductive or osteogenic action, for example in use associated with demineralized bovine or human bone matrix or auto or allogenic.

The use of Enxerto Ósseo Orgânico Baumer (GenOx Org)is not recommended in any medical or surgical condition that may compromise the success of the surgical procedure.

#### STERILITY

The medical product Enxerto Ósseo Orgânico Baumer (GenOx Org) is marketed STERILE. The product is sterilized with gamma radiation.

Non used product must not be reused.

Enxerto Ósseo Orgânico Baumer (GenOx Org) must not be re-sterilized.

#### WARRANTY

The guarantee will be applied to medical products as long as it is used in accordance with the previous sections of this instruction for use: FUNDAMENTALS OF THE PRODUCT OPERATION AND ACTION, CONTENTS AND PRESENTATION, SUPPORT MATERIALS, INDICATIONS, IMPORTANT INFORMATION, TECHNICAL AND OPERATIONAL PREPARATION FOR USING THE PRODUCT, INSTRUCTIONS FOR USE, INSTRUCTIONS AND PROCEDURES FOR DISPOSAL OF THE PRODUCT, PRECAUTIONS, WARNING, POSSIBLE AADVERSE EFFECTS, SPECIAL CARE AND CLARIFICATIONS ABOUT STORAGE AND TRANSPORTATION, CONTRAINDICATIONS, AND STERILITY.

#### COMPLAINT

Any customer or user of these medical products who wishes to claim, if he is not satisfied with the services and / or products offered by BAUMER, such as: identification, traceability, security, effectiveness, performance, should contact the authorized BAUMER distributor.

Any problems or suspected problems with related medical products should be reported immediately to BAUMER or your authorized BAUMER distributor.

Any serious adverse effects that affect the health or safety of the patient or user, such as malfunction or damage to medical products, serious problems or death caused by, or associated with, the use of these products, must be reported to BAUMER. In the event of any adverse effect related to the product that affects the health or safety of the patient or user, the dentist or physician must notify the competent health authority (ANVISA through the website http://portal.anvisa.gov.br and report what happened to BAUMER or the authorized BAUMER distributor.

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